

# JUXTA-FIT™

## CUSTOM GLOVE

### COMPONENTS OF JUXTA-FIT™ GLOVE:

Nylon, Polyurethane, Spandex, Silver

**COMPONENTS OF LABEL:** Polyvinyl Chloride Plastisol or Satin Acetate

**THIS PRODUCT CONTAINS SILVER.**

**THIS PRODUCT DOES NOT CONTAIN LATEX.**

**INDICATIONS FOR USE:** Lymphedema.

**CAUTIONS:** Decreased or absent sensation in the hand, Sensitivity to product components.

**CONTRAINDICATIONS:** Severe arterial insufficiency, acute infection.

Custom garments require a prescription. This garment is not designed to be cut or modified in anyway. Doing so will void the six month limited warranty. CircAid Medical Products, Inc. assumes no liability for injury or damages resulting from contraindicated use of its products. CircAid is a registered trademark and Juxta-Fit is a trademark of CircAid Medical Products, Inc. ©2012 CircAid Medical Products, Inc. All Rights Reserved.



CircAid Medical Products, Inc.  
9323 Chesapeake Drive, Suite B2  
San Diego, CA 92123 U.S.A.  
01-858-576-3550

EC

REP

MDSS  
Schiffgraben 41  
30175 Hannover, Germany

**CIRCAID**  
by medi

JF-GL-DFU-112612-01

## DIRECTIONS FOR USE:

### DONNING THE JUXTA-FIT™ GLOVE:

**STEP 1.** Once your compression garment is in its proper position and secured, slide the Juxta-Fit™ Glove over the hand.

**STEP 2.** Ensure the cuff portion is pulled over the wrist section of the compression garment.

**STEP 3.** Fasten the wrist band on the glove. It should be snugly attached with very light compression.

**NOTE:** It is important not to over tighten the glove wrist band so that fluid can freely move up and away from the hand.

### REMOVING THE JUXTA-FIT™ GLOVE:

**STEP 1.** To remove the glove, first detach the wrist band and fold it back onto its self.

**STEP 2.** Working with one finger at a time, slowly work them out of the glove by pulling on the edge of each finger. Continue until the glove easily slides off the hand.

**Note:** The Juxta-Fit™ Glove should never hurt. It should feel firm but comfortable. If pressure increases or decreases noticeably during wear, loosen or tighten the bands accordingly.



**If you are experiencing any pain, immediately remove the Juxta-Fit™ Glove and consult your physician.**

## CARE INSTRUCTIONS:



**HAND WASH\***



**DRIP DRY OR FLAT DRY\***



**DO NOT BLEACH**



**DO NOT IRON**



**DO NOT DRY CLEAN**



**DO NOT WRING**

\*To extend the life of your Juxta-Fit™ Glove hand wash and drip or flat dry. The Juxta-Fit™ Glove can also be machine washed using a gentle cycle and dried on a low heat setting.



**STEP 1**



**STEP 2**



**STEP 3**

